Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

Triton

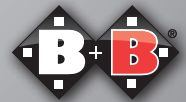
DE

EN

ES

FR

Enjoy mobility.

Inhalt

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Index

1.	Preliminary remark	36	4.7.	Backrest height	47
1.1.	Signs and symbols	36	4.8.	Seat angle	48
2.	Important safety instructions	37	4.9.	Backrest angle	48
2.1.	General safety instructions	37	4.10.	Locking/parking brake	49
2.2.	Brakes	38	4.11.	Push handles	50
2.3.	Anti-tip wheels	38	4.12.	Trunk support pads	50
2.4.	Tyres	38	4.13.	Tip protection	50
2.5.	Obstacles	39	4.14.	Head rest	51
2.6.	Up and down gradients	39	4.15.	Therapy table	52
2.7.	Safety in vehicles	39	4.16.	Lap belt (optional)	52
2.8.	Use on Public Roads	39	5.	Use	53
2.9.	Purpose	39	5.1.	Entering and leaving from the	53
2.10.	Indications	40	5.2.	Entering and leaving from the	53
2.11.	Contraindications	40	5.3.	Tipping lever	54
2.12.	Responsibility	40	5.4.	Negotiating stairs and high	54
2.13.	Declaration of conformity	41	5.5.	Drum brake (optional)	55
2.14.	Service life	41	5.6.	Transporting the wheelchair	56
3.	Description of product and	41	6.	Technical data	57
3.1.	Inspection on delivery	41	7.	Transport	59
3.2.	Overview and contents	42	8.	Maintenance tips for users	60
3.3.	Type plate and serial number	43	8.1.	Cleaning	60
4.	Using the wheelchair	44	8.2.	Inspection of the wheelchair by the user	60
4.1.	Set up	44	9.	Maintenance Tips for Retailers	61
4.2.	Foot plates	45	10.	Use by others	61
4.3.	Leg rests	45	11.	Storage / dispatch	62
4.4.	Side panels and armrest	46	12.	Disposal	62
4.5.	Seat width	47	13.	Warranty	63
4.6.	Seat depth	47			

1. Preliminary remark

Dear User

You have chosen a high-quality wheelchair from Bischoff & Bischoff. Thank you for your trust in our product.

The wheelchair was designed to offer you a lot of benefits and to meet your needs. The Triton wheelchair allows you to adjust the chair's settings to your individual body size and shape thanks to the variety of adjustment options and optimum adjustability of the seat depth, backrest height, knee angle, seat width, backrest angle, seat angle and the length of lower leg. Triton is intended both for indoor and outdoor use.

Before using your new electric wheelchair for the first time, please read and follow the instructions in this manual carefully. The instruction manual is an integral and necessary part of the wheelchair. This is why you should keep it close by and include it when passing on the wheelchair to someone else.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized B+B dealers.

1.1. Signs and symbols

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

Important! This is especially useful information on a given subject.

2. Important safety instructions

2.1. General safety instructions

Please make sure that all parts have been properly installed before using the wheelchair.

To avoid falls and dangerous situations, you should first practice driving your new wheelchair on level, unobstructed ground. We recommend having an assistant with you.

- The wheelchair may not be used for transport of more than one person or any additional load. Make sure to only use it for the intended purpose.
- Never use the wheelchair under the influence of alcohol or other drugs.
- Please note that displacing the center of gravity by shifting the body or by adding loads may increase the risk of tipping over the wheelchair to the side or backwards.
- Please note, the upholstery will heat up if directly exposed to sunlight and may cause injuries to your skin when touched. Therefore you should cover these parts or protect the wheelchair from exposure to direct sunlight.
- We recommend leather gloves for outdoor use. These increase grip and protect fingers and hands from dirt and injury at the same time.

2.2. Brakes

Parking brake

The brakes, which work directly on the tyres of the wheels, are only to be used as parking brakes, which means they must only be applied when the wheelchair is standing still, not for reducing the speed. When braking at fast speeds or on long downward slopes when using the push rims, fingers and hand surfaces become hot.

Caution! Danger of burning!

Drum brake (optional)

The drum brake is exclusively intended for operation by the individual pushing the wheelchair. The drum brake can be used to secure the wheel chair as well as for controlled braking.

2.3. Anti-tip wheels

The anti-tip wheels prevent the wheelchair from tipping backwards. Please note, driving with anti-tip wheels on uneven ground, lift platforms and ramps is only possible to a limited extent. Ensure sufficient clearance above, below and on the sides.

2.4. Tyres

Always ensure the tyres have a sufficient tread depth (> 1mm). If the tread depth is too small, it is no longer possible to guarantee safe handling of the wheelchair.

2.5. Obstacles

Please always use ramps to drive over obstacles (e.g. steps, kerbs, etc.). Avoid driving into an obstacle without braking or jumping down from one level to another.

2.6. Up and down gradients

Always be aware that the backrest of your wheelchair may tip backwards at extreme settings, even at a slight inclination. When extreme settings have been applied, only navigate inclinations and gradients with the support of an accompanying person. Also use the anti-tip wheels to prevent tipping over backwards.

2.7. Safety in vehicles

Please note that wheelchairs should not be used as a seat in any vehicles. Users must change from the wheelchair to a vehicle seat.

2.8. Use on Public Roads

The wheelchair is designed for indoor and outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations.

2.9. Purpose

The wheelchair is designed solely to increase mobility and to transport people with impaired mobility in accordance with indications listed. The maximum user load is ≤125 kg.

2.10. Indications

Inability to walk or severe walking disability as a result of:

- paralysis
- loss of limbs
- meromelia/limb deformation
- joint contractures/joint damage
- other ailments

The wheelchair has been equipped with the option of a backrest adjustment setting range of 30° and foot rests which can be adjusted upwards until horizontal, which should be used if almost horizontal alignment is intermittently required, e.g. as a result of orthostatic collapse conditions.

2.11. Contraindications

Provision of wheelchairs is not suitable for people:

- who are unable to sit
- with joint damage/joint contracture of both arms
- with perceptual disorders
- with severe equilibrium disorders
- with reduced and inadequate vision
- with serious cognitive restrictions

2.12. Responsibility

We are only able to provide a warranty if the product is used in accordance with the specified conditions and for the intended purposes. If modifications, repairs or maintenance works are made, please make sure only authorised persons perform these works in accordance to the instructions of B+B.

2.13. Declaration of conformity

B+B, as manufacturer and solely responsible party, hereby declares that the wheelchair Triton complies with the requirements of the EC Directive 93/42.

2.14. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the wheelchair can be used continuously, provided it is in a safe condition.

3. Description of product and contents

3.1. Inspection on delivery

The Triton wheelchair is delivered fully assembled in cardboard box. After unpacking, please retain the carton if possible. It can be used to store or return the wheelchair at a later stage.

Delivery of the wheelchair, instructions on its use and adjustments to the chair are carried out by qualified and authorised medical supply stores.

If shipped by rail or a forwarding agent, the goods are to be checked with regard to transportation damage (carton) immediately in the presence of the delivery company. If it is damaged, please contact our customer service department immediately:

Tel.: +49(0)700/6000 7070).

Check that the contents are complete and intact. In the event of irregularities and damage, please contact our customer service department by telephoning: Tel.: +49(0)700/6000 7070

3.2. Overview and contents

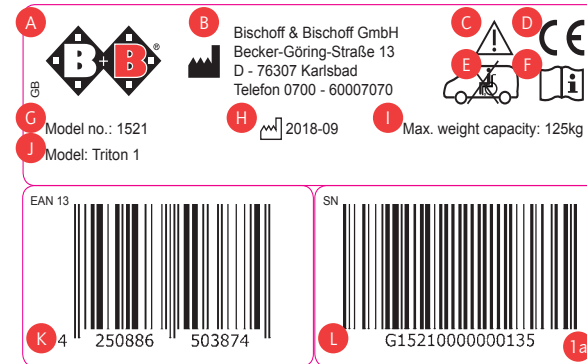
The content consists of:

- Wheelchair, pre-assembled
- Pair of legrests (angle adjustable)
- 1 Head rest
- 1 Pair of side panels with armrests
- 1 Pair of side pads/trunk supports
- 1 Pair of calf and knee pads

3.3. Type plate and serial number

The type plate (fig. 1 a) and serial number are located on the bottom of the frame.

- A** Manufacturer logo
- B** Manufacturer information
- C** Caution! - Follow instructions for use
- D** CE mark
- E** Follow instructions regarding securing the product in the vehicle
- F** Important - Adhere to the operating instructions
- G** Model designation
- H** Date of manufacture
- I** Maximum weight capacity
- J** Model number
- K** EAN 13 item number
- L** Serial number

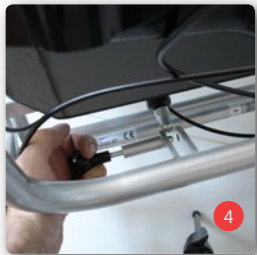
4. Using the wheelchair


4.1. Set up

After unpacking the box, you must first put the wheelchair in working order and get it ready for use.

- Put the back rest in the upright position (Fig. 3).
- Lock the backrest against the gas-pressure cylinder in order to adjust the backrest (Fig. 4).
- Fasten the Bowden cables by loosening the bracket (Fig. 5) from the armrest using the Allen key. Position the Bowden cable underneath the bracket and re-tighten the hexagon socket screw.
- Put the seat cushion and padding in place.

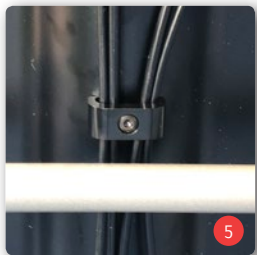
Position the hook and loop tape and Velcro strap directly over each other.

- Put the armrests with the rails in the holder. The adjusting knob below the armrest must be at the front. Press the button on the holder (Fig. 6) in such a way that you can insert the armrest rails. Once you have released the button, the rails will audibly click into place.
- Attach the drive wheels with the quick-release axle. (Fig. 7)
- Activate the parking brakes on the right and left drive wheels.
- Mount the leg rests and fold the foot plates upwards.
- Add the headrest.
- Now adjust the trunk support pads, lower leg rests and headrests.

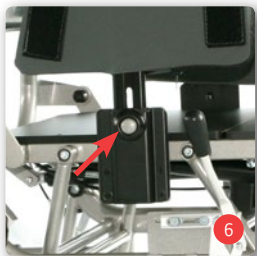RISK OF ENTRAPMENT! Ensure that you do not trap your fingers when folding down the backrest.

4.2. Food plates

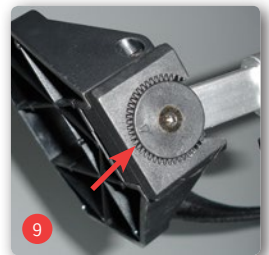
- The length of the support for the lower leg is adjustable in 5 steps (each 22,5cm).
- Unfasten the plastic screw (Fig. 8).
- Select the desired height.
- Re-tighten the plastic screw.
- The foot plates must be folded upwards and swung to the side when getting into or out of the wheelchair.
- The footplate can be adjusted along the notches in the bracket by loosening the Allen (hexagon) screw (SW 5) (Fig. 9).

Caution! The markings will make it easier for you to position the mounting holes.

4.3. Leg rests

- You can fold the foot plates upwards, or swing the legrests outwards and take them out by pulling upwards so that you can get in and out of the wheelchair as easily as possible.
- In order to fold the leg rests outward and away, pull them upward by approx. 3 cm until it is possible to fold them outward and away. (Fig. 10).
- In the "outer position", the leg rests can be completely pulled out upwards (Fig. 10).
- To assemble the leg rests, proceed in the reverse order.
- Push the tube back into the leg rest bracket.
- Fold the leg rest back to the front until it slides back to its original position.
- The legrests are independent from each other and can be adjusted until they are in a horizontal position.


- Pull the leg rests until they are in the desired position.
- The leg rests will automatically remain in the selected position.
- In order to lower the leg rests again, operate the lever (Fig. 11).
- The calf pad can be moved up and down the leg rest.
- Loosen the fixing screws (SW4) and place the calf pad in the desired position (Fig. 12).
- Tighten the fixing screws again.


4.4. Side panels and armrest

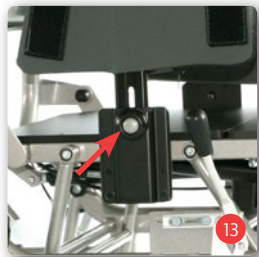- The height of the side panels, including the padded armrest, can be adjusted.
- In order to do so, operate the locking button and adjust to the desired height (Fig. 13).
- In order to make mounting and dismounting easier, you can also remove the side panels completely.
- The padded armrest can also be adjusted horizontally.
- Loosen the plastic screw underneath the padding of the armrest (Fig 14).
- Move the armrest to the desired position and tighten the screw again.


4.5. Seat width

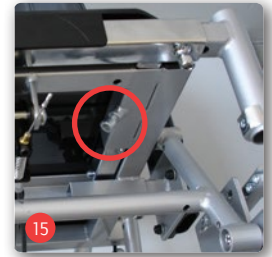- The width of the seat can be extended for both models by 12 cm each in order to meet the needs of the user.
- The same adjustments must be made on the right and left side.
- In order to adapt the seat width, the leg supports and the side panels must be moved.
- Tip the wheelchair onto the push handles.
- Loosen the socket head screw (SW 6) (Fig. 15).
- Adjust the seat to the desired width and re-tighten the socket head screws.

4.6. Seat depth

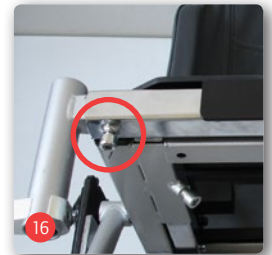- The depth of the seat can be adjusted by 12 mm.
- Take the seat padding off.
- Loosen the 4 Allen hexagon screws (SW4) on the edge of the seat (Fig. 16).
- Now, move the seat to the desired position and re-tighten the screws again.
- Place the seat padding back on the seat.

4.7. Backrest height

- The back height can be adjusted by 16 cm in both versions..
- Take the backrest padding off.
- Loosen the plastic screws at the back. (Fig. 17)
- Adjust the backrest to the desired height and tighten the screws again.
- Put the padding for the backrest back on the wheelchair.

4.8. Seat angle

- The seat angle of the Triton wheelchair can be variably adjusted from 0° bis 15° (Fig. 19).
- However, the angle between the backrest and the seat will remain unchanged.
- On the right push handle there is a release lever which must be used to adjust the seat angle (Fig. 20).
- The gas pressure cylinder supports the adjustment of the seat angle.
- Let go of the release lever and the position of the seat angle will remain in place.

4.9. Backrest angle

- The backrest angle of the Triton wheelchair can be variably adjusted backwards from 3° - 43° (Fig. 21).
- However, the angle of the seat to the ground will remain unchanged.
- On the left push handle there is a release lever which must be used to adjust the seat angle (Fig. 22).
- The gas pressure cylinder supports the adjustment of the backrest angle.
- When you let go of the release lever, the position of the seat angle will remain in place.

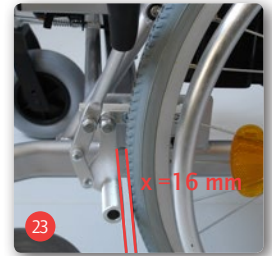
4.10. Locking/parking brake

The Triton wheelchair is equipped with a wheel lock, which can be controlled by the person sitting in the wheelchair. It must only be used to park the wheelchair. It is not suitable for slowing down the wheelchair while it is in motion! The push handles should be used for this purpose by applying light pressure on the push handles with the palm of your hands.

After the wheelchair is slowed down this way, you can then apply both wheel locks by pushing the brake levers forward. The wheelchair is now safely held in place.

The distance between the pivot bolt and the tyre cover must be X= 16 mm at the narrowest point when the braking mechanism is opened to its maximum position (Fig. 23).

- To adjust the distance, loosen the two fixing screws on the brake mounting.
- You can then slide the braking system along the mounting into the correct position.
- The braking bolt must be aligned horizontally so that it is parallel to the ground surface.
- Then re-tighten both Allen hexagon screws again.
- Make sure that the brake fully locks into place when the lever is operated and that it does not snap back.

To ensure the brake operates to its maximum capability, the tyre pressure must be 300 - 400 kPa (3-4 bar) for pneumatic tyres. If the tyre pressure is not as specified, there is a loss of braking efficiency.

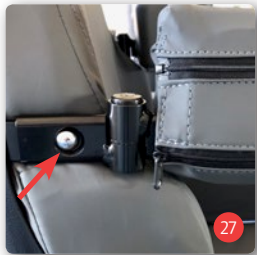
4.11. Push handles

- The Triton push handle can be continuously adjusted upward by 17 cm.
- Loosen the plastic screws to the rear on the back tube.
- Adjust the push handle to the desired height and tighten the socket head screws again.
- At the crossbar between the handles, you will find the release lever to adjust the seat and backrest angle (Fig. 24).

4.12. Trunk support pads

- The height, width and angle of the side of the Triton's trunk support pads can be adjusted.
- Loosen the upper plastic screws at the back (Fig. 25).
- Position the trunk support pads at the desired height and width of the chest.
- Re-tighten the plastic screw.
- You can adjust the angle by pulling the trunk support pad upward until the teeth are disengaged. (Fig. 26).
- Position the trunk support pads at the desired angle to the chest.
- Release the trunk support pad again; it will click into place automatically.
- In order to remove the pad, hold the button pushed in and pull the pad forwards out of the bracket (Fig. 27).


4.13. Tip protection

- The anti-tipping protectors prevent the wheelchair from tipping over backwards. There must be a gap of 3-5 cm between the anti-tip wheels and the ground.
- It is highly recommended that the anti-tippers be used when the wheelchair is being operated by inexperienced users or by wheelchair users with an amputated leg.

- The anti-tippers are attached as a pair to the right-hand and left-hand sides of the frame.
- The height of the anti-tippers can be adjusted.
- Pull the metal disc backwards and slide the anti-tipper into the desired position (Fig. 28).
- Release the disc once again.
- The wheel will now click firmly into place in the next possible position.



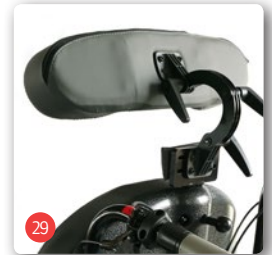
Always use the anti-tip wheels in pairs.

4.14. Head rest

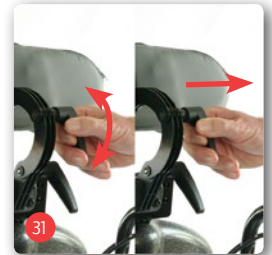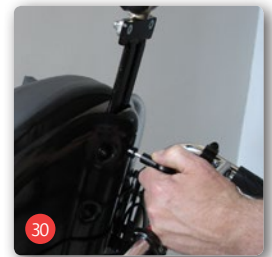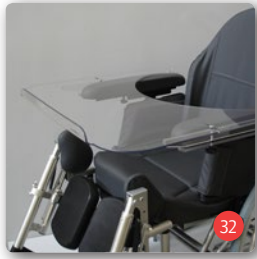- The Triton wheelchair is equipped with a headrest; its angle, height and depth can be adjusted. (Abb. 29).
- Pull the locking mechanism on the backrest out (Fig. 30).
- Position the headrest at the desired height and then let go of the locking mechanism.
- The slide bar will snap into place in the next possible position.
- With the help of the quick-release lever in the neck and head area of the headrest, the depth and angle of the headrest can be adjusted (Fig. 31).
- Release the screw connection by pulling the lever upward.
- Position the headrest at the required depth.
- Re-tighten the screw connection with the help of the lever.
- In order to bring the lever to another position without loosening the screw, pull the lever to the side; it can therefore be adjusted without moving the screw.

4.15. Therapy table

The Triton wheelchair can be equipped with a therapy tray (Fig. 32).

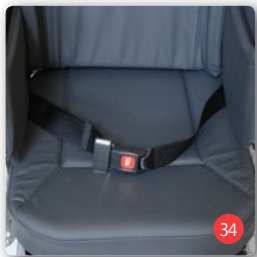
- Loosen the knob screws on the therapy tray.
- Position the mounting rails at the maximum width.
- Place the therapy tray on the arm pads and push the bracket against the armrest to the right and to the left.
- Secure the therapy tray by tightly screwing on the knob screws (Fig. 33).
- The height of the tray can be adjusted. To do this, please consult the instructions on adjusting the height of the side panels.
- You can adjust horizontally in the same way as when you move the arm pad.

4.16. Lap belt (optional)

There is also an optional lap belt available; it secures wheelchair users who do not have the stability required for sitting in a wheelchair (Fig. 34).

- The Triton wheelchair can be equipped with a seat belt for the pelvis.
- Use the buckle to open and close the belt.
- The length of the belt can be adjusted using the movable latch on the belt.

5. Use

5.1. Entering and leaving from the side

- Move the wheelchair as close as possible to the present or future seat so they are side by side.
- If you wish to move from another wheelchair or room chair, lock it by setting the parking brakes.
- Secure the Triton against rolling away accidentally by activating the parking brake.
- Pivot the armrest backwards and away, or remove the side part with the armrest.
- Fold the foot plates/footrest to the side or remove the leg rests.
- Now slide sideways onto the other seat.
- Make sure that you are sitting as far back in the seat as possible.
- Finally, return the leg rest to their original position.

Never stand on the footrest!
Risk of tipping

5.2. Entering and leaving from the front

- First fold the foot plates up or remove the leg rests.
- Move the Triton and the current or intended seat together front to front if possible and decelerate the Triton
- If you wish to move from another wheelchair or room chair, lock it by setting the parking brakes.
- Now slide onto the seat by turning your body around.
- Then move the arm rest and leg rests to the original position.

5.3. Tipping lever

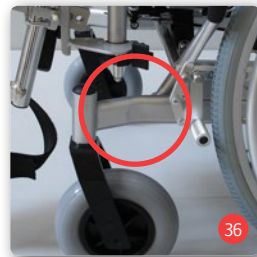
- The tipping lever makes it easier for an assistant to tilt the wheelchair slightly backwards in order to go over a step for example.
- To do this, push the lever down using your foot (Fig. 35).
- Use the push handles to keep your balance.
- The anti-tip wheels are installed with integrated tip-assist function.


5.4. Negotiating stairs and high steps

For obstacles and stairs with more than three steps, two accompanying people shall help you.

- The helpers shall take hold only at securely mounted parts of the frame (see Fig. 36 and 37).
- When going up any stairs, the helper behind the wheelchair pulls the wheelchair over the step, holding it at the securely mounted handles.
- The helper who is below takes hold of the front frame tubes and stabilises the position. At the same time, he pushes the rear wheels of the wheelchair against the steps.
- When going down any stairs, the helper who is below brakes by pushing the wheelchair right against the steps. The helper above holds the wheelchair at the push handles, secures it and holds it in the right position. The wheelchair should roll down step by step

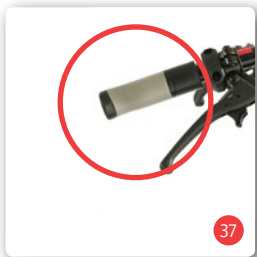5.5. Drum brake (optional)

As an option, the Triton can be equipped with a drum brake, which should only be operated by an attendant person via the brake lever on the push handles.

The drum brake should be used as both a locking brake and also as a service brake. When used as a service brake, it provides well-regulated deceleration. The locking brake function is used for parking. To lock the brake, simply pull the brake lever and push the locking lever upwards (see fig. 38).

- To release the locking brake again, simply pull the brake lever briefly. (The locking mechanism releases automatically).

To achieve optimum braking efficiency, the braking power is set by using the adjusting screw (see fig. 39). The braking power is increased by unscrewing the adjusting screw and reduced by tightening it.

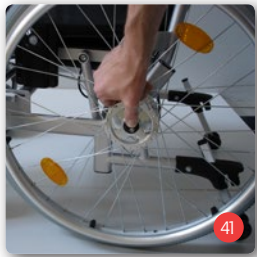
- Then tighten the adjusting screw until the friction noises disappear. The wheel then runs freely.
- After completing the setting, the adjusting screw is fixed by tightening the locknut.

5.6. Transporting the wheelchair

- In order to prepare your wheelchair for transportation in a car, proceed as follows.
- Remove the leg rests (Fig. 40).
- Release the backrest lock on the gas-pressure cylinder.
- Fold the backrest forward.
- Release the parking brakes on the right and left of the drive wheels.
- Now you can easily remove the rear wheels via the quick-release axle (Fig. 41).

The frame and the push handles are the only places where the wheelchair can be gripped to transport it. Armrests must never be used to lift the wheelchair.

Do not use the side panels to carry the wheelchair!

6. Technical data

Dimensions

Max. user load:	125 kg
Seat width:	36 - 43 (Var. I) / 43 - 50 (Var. II) cm
Seat depth:	42 - 48 cm
Seat height:	52 cm
Seat plane angle:	0 to 15° (continuous)
Overall width:	68 (Var. I) / 73 (Var. II) cm
Overall length:	104 / 84 cm (with/without Legrest)
Overall height:	90 - 120 cm
Backrest height:	50 - 62 cm
Lower leg length:	40 - 49 cm (5x)
Backrest angle:	-3 - 43° (without adjustment)
Food plate angle:	0 - 25° (5-x)
Knee angle:	90 - 180° (without adjustment)
Height of the armrest:	22 - 28 cm (5-fach)
Length of the armrest:	42 cm (7,5 cm adjustable)
Clearance between the armrest and the seat surface:	31 - 37 cm
Empty weight:	38,7 kg (Var. I) / 40,2 kg (Var. II)
Frame weight:	24,7 kg
Dimensions when folded:	84 / 73 / 65 cm (L/W/H)
Min. turning radius:	240 cm
Camber:	0°
Static stability downhill / uphill:	14,3° / 18.2°

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Material

Frame: Steel profile tube powder coated
 Attachments: Aluminium tube powder coated
 Seat, back: Foam, Dartex
 Armrest cushion: Rigid foam, Dartex
 Flame-retardant upholstery in accordance with (EN 1021-1 and -2): yes

7. Transport

You can determine whether or not your wheelchair is approved for use as a seat in a vehicle for transporting disabled people by referring to the symbols on the CE sticker on the product:

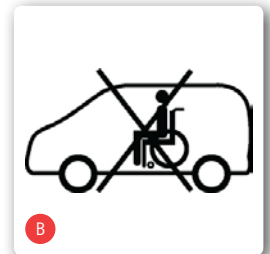
Wheelchair suitable for use as a seat in a vehicle for transporting disabled people (fig. A)

Wheelchair not suitable for use as a seat in a vehicle for transporting disabled people (fig. B)

All products approved by B+B for use as seats in vehicles are tested in accordance with ISO 7176-19. The tests in accordance with ISO 7176-19 were performed using restraint systems from AMF-Bruns. The correct mounting of the restraint system is a prerequisite for the safe transportation of the products in a vehicle for transporting disabled people and the basis for the approval.

Fixation points on the restraint system (fig. C)

The restraint systems from AMF-Bruns provide the ideal points for the provision of restraint force for the purpose of securing the products in the vehicle for transporting disabled people (known by the German term „Kraftknoten“, which is defined in the standard DIN 75078-2). For the purpose of providing restraint in the vehicle for transporting disabled people (fig. C), the products must only be secured at these specific points („Kraftknoten“ points). If possible, use a permanently installed car seat and its belt system. Stow the wheelchair in the luggage space. The person responsible for the transportation should be familiar with the current regulations for transportation as well as with the method of operating the vehicle for transporting disabled people and the restraint system. Anchored 4-point restraint systems must be used to secure the wheelchair in the vehicle. If you have any queries about securing your product in a vehicle for transporting disabled people, please contact the B+B customer service department on +49 (0)700/60007070.

8. Maintenance tips for users

8.1. Cleaning

- The frame and the paneling of the wheelchair can be cleaned with a moist cloth. You can use a mild detergent for more stubborn dirt
- The wheels can be cleaned with a wet synthetic fiber brush (do not use a wire brush!)
- Push handles, arm pads, and seat and back pads can be washed off. Use mild detergent.

Caution! Do not use a high-pressure or steam cleaning device

8.2. Inspection of the wheelchair by the user

Check these items before beginning each journey

- Make sure that the tyres are in good condition. Carry out a regular check of the tread depth of the tyres once a month. This should be at least 1 mm. Any faulty or worn out tyres must be changed.
- Ensure that the parking brakes function properly. When the brake is applied, the wheelchair should not be able to be moved. When the brake is released, the wheelchair should be able to be moved around easily without making any friction noises and it should have good directional stability. If you notice unusual braking performance, inform your specialist dealer immediately and do not use the wheelchair.
- Make sure the seat cover and back cover are in good condition and clean. If the covers get dirty, you can clean them according to the cleaning instructions. If they are damaged, contact your specialist dealer.
- Check the folding mechanism and all of the movable parts and

ensure that they can be moved easily and work without making a lot of noise. Squeaking noises indicate that there is damage and your wheelchair should be checked by a specialist dealer.

- Make sure that all screwed fittings and riveted joints are intact and in good condition

9. Maintenance Tips for Retailers

We recommend to have the entire wheelchair inspected by a qualified mobility dealer at least once a year. Should the wheelchair be faulty or defective, it should be brought to a qualified mobility dealer for repair without delay.

Repair and replacement of parts are carried out by the mobility dealer. All information, documents and schematics necessary for repair are made available to a qualified mobility dealer on request. There is also a training program for authorized dealers.

Should you have any questions, please contact the B+B customer service at +49 700 - 6000 7070

The wheelchair shall be inspected in accordance to the service schedule and must be in good condition

10. Use by others

The Triton is appropriate for use by others. For use by others, the wheelchair must first be thoroughly cleaned and disinfected according to the accompanying care and hygiene instructions. The product shall get checked by an authorised mobility dealer for general condition, wear and shall get repaired if necessary.

11. Storage / dispatch

The wheelchair is going to be stored or transported, all removable and not tightened parts, except of the drive wheels, have to be removed and packed in separate cartons. The separately packed parts can be packed together in a larger carton.

It is recommendable to keep the original packaging for this purpose, so that it is available in case you need it. In that way your wheelchair is protected against environmental influences during storage or transport.

12. Disposal

If your wheelchair is no longer needed and you want to dispose it, please contact your qualified mobility dealer. If you want to dispose your wheelchair yourself, please ask your local disposal company for the applicable disposal regulations in your town or city.

13. Warranty

Guarantee items refer to all defects of the wheelchair which can demonstrably be referred to material defect or manufacturing defect. If there are complaints you have to provide a completed certificate of guarantee.

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

We provide the statutory warranty of 2 years for our products. For batteries, replacement devices and repairs, a warranty period of 1 year applies.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

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Bischoff & Bischoff GmbH
B+B France S.A.R.L.